


INSTRUCTION

FFP2 - SNN20647, SNN70370B, SNN70369B - CE2834 - FFP2 NR

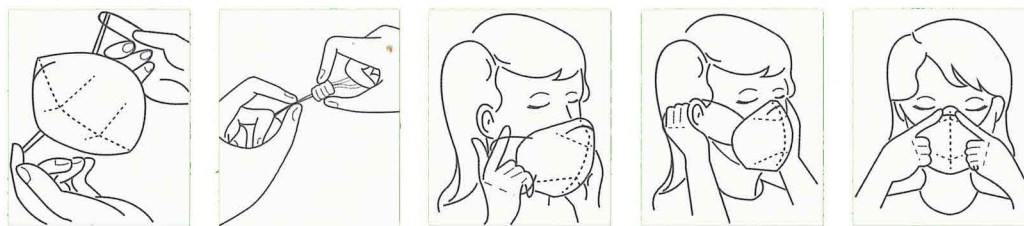

- ① Place fingers inside the particle half mask. Bend the nose clip around fingers to form nose shape.
- ② Hold the particle half mask in position over the nose and mouth, pull the ear loop behind the ears.
- ③ Using both hands, Adjust the nose clip and make a fitting curve inside. Shift the ear loop to ensure a positive face seal.
- ④ Fit check:
 - To test the fit of the particle half mask without an exhalation valve, cup both hands over mask and inhale sharply.
 - If air flow is felt in the nose area, re-adjust/tighten the nose clip.
 - If air flow is felt around the edges of the mask, re-position the particle half mask / ear loop to achieve a better fit.
- ⑤ Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
- ⑥ Change the mask if a proper face seal cannot be achieved.
- ⑦ **Recommended usage time: 4-8 hours.**
- ⑧ Careful observance of these instruction is an important step in safe mask use.

USE LIMITATION

- ① Do not use the mask or enter or stay in a contaminated area under the following circumstance.
 - Atmosphere contains less than 19.5% oxygen.
 - If you smell or taste contaminant.
 - For protection against gases or vapors.
 - Contaminants or their concentrations are unknown or immediately dangerous to life or health.
 - For sandblasting, paint-spray operations and asbestos.
 - In explosive atmospheres.
- ② Do not modify or misuse the mask.
- ③ Do not use the particle half mask with facial hair or any other conditions that may prevent a good face-seal as the requirements for leakage will be achieved.
- ④ Particle filtering half mask needs to be inspected prior to each use to assure there are no holes in the breathing zone other than punctures around and staples and no damaged has occurred. Enlarged holes resulting from ripped or torn filter material around staple punctures are considered damage.
- ⑤ This mask helps protect against certain particulate contaminants but does not eliminate exposure to risk or contracting disease or infection. Misuse may result in sickness or death.
- ⑥ This particle half mask marked "NR", shall not be used for more than one shift.
- ⑦ This particle half mask is not suitable for users who have damaged ears.